

Cleaning Agent	Stafmat 5099	CPU150	CPU150 Ultra	China CPU150	LVT	PGN71	PUN27	PUN24	PUN71	PVC Nonskid	RSDTEX	TPU NS	70D HMVT	200D HMVT
CSI Disinfectant Cleaner	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Virex ® II 256 One-step Disinfectant Cleaner & Deodorant	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Wex-cide 128	YES	No	No	No	Yes	Yes	No	No	No	Yes	Yes	Yes	No	No
Dispatch ® Hospital cleaner disinfectant towels with Bleach	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Forward DC	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Amphyl ® Brand III - Disinfectant spray	YES	No	No	No	No	No	No	No	No	No	No	No	No	No
Virex II 128 One-step Disinfectant Cleaner	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Lysol ® IC ™ - Quaternary Disinfectant Cleaner Concentrate	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Lysol ® IC ™ Disinfectant Spray	NO	No	No	No	No	No	No	No	No	No	No	No	No	No
Vesphene ® II	YES	No	No	No	No	No	No	No	No	No	No	No	No	No
1-Stroke Environ ®	YES	No	No	No	No	No	No	No	No	No	No	No	No	No
Omega Hi-Chlor Solid Machine Detergent	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Sanikleen	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
HB Quat	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Quat Sanitizer II	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
A-33	NO	Yes	Yes	No	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Super Sani Cloth Germicidal Disposable Wipe	NO	Yes	Yes	No	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Elimstaph #2	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Sani-Cloth Plus Germicidal Disposable Cloth	NO	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Solid Surge Plus	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
Spitfire SC Power Cleaner	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
#1024	Yes but only to wipe, then rinse thoroughly. Do not submerge in liquid combined with this cleaning agent.													
#1204	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
BNC-15	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
Clorox Ultra Germicidal Bleach	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
Microban X-590	Yes but only to wipe, then rinse thoroughly. Do not submerge in liquid combined with this cleaning agent.													
Oxgen Detergent	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
Stain Away	Yes but only to wipe, then rinse thoroughly. Do not submerge in liquid combined with this cleaning agent.													
Steri Fab	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
CR85336	NO	NO	NO	NO	Yes - wipe only**	Yes - wipe only**	NO	NO	NO	Yes - wipe only**	NO	NO	NO	NO

NOTE:

The above list is not intended to be all inclusive nor should it be considered a recommendation or endorsement by Stafford Textiles Limited on any product. Please refer to our cleaning recommendations sheet for instructions.

It is extremely important to ALWAYS rinse the surface of the fabric AFTER cleaning with any compatible cleaning agent, regardless of agent instructions. Not rinsing may result in premature degradation of the fabric.

As a guideline, Phenolic disinfectants are the best choice for Vinyl laminates, though properly diluted quaternaries are also acceptable.

Generally, quaternary disinfectants are recommended for the Nylon/Urethane and Nylon/PVC fabrics. Phenolics should be avoided on the Nylon/Urethane and Nylon/PVC Fabrics.

Quaternary/Isopropyl disinfectants ARE also recommended for the Nylon/Urethanes and Nylon/PVC fabrics but NOT recommended for VINYL LAMINATES.

Applying any cleaning agent in high concentrations will decrease the useful life of Stafford Textiles Limited's fabrics. Iodophor disinfectants may result in staining.

** - note - wipe only means the following: Yes but only to wipe, then rinse thoroughly. Do not submerge in liquid combined with this cleaning agent.

This information is believed to be reliable and offered in good faith and intended for use as a general guide. Stafford Textiles Limited makes no guarantee of results and assumes no obligation or liability whatsoever in connection with the use of this information. This information, including any statement concerning the possible use of our products or any other manufacturer's products, is not a license to operate under or intended to suggest infringement of any existing patents.