

# Cleaning Recommendations

Product: CPU150 Healthcare Fabric  
Document #11.VCR.0015.01  
Revision Date: June 2017



---

## Routine care:

- Remove all gross contamination from the surface.
- Wipe fabric clean with neutral soapsuds and lukewarm water.
- Rinse thoroughly and allow to dry before return to service.
- **Hard to Clean Areas:** Soft sponge with approved liquid cleaner diluted to chemical manufacturer's recommendations. **NOTE:** Do not use harsh cleaners or solvents. Follow all manufacturer directions for dilution of concentrated cleaning agents. For a list of approved cleaners/disinfectants, please visit the FAQ section at [www.5mindsmobility.com](http://www.5mindsmobility.com).
- To control or prevent odor on long-term incontinent applications, clean CPU150 daily. The use of scented cleaner/disinfectant is recommended.

## Disinfection:

- Clean first, as described above.
- Most phenolic or quaternary type disinfectants can be used on CPU150 fabric. User must dilute disinfectants and germicides in accordance with manufacturer's instructions.
- Iodophor type disinfectants (Betadine, for example) will stain CPU150 fabric.
- Rinse thoroughly and allow to dry before return to service.

## Blood Contamination:

- CPU150 can be disinfected with 1:10 dilution of household bleach (5.25% sodium hypochlorite) as recommended by the CDC (U.S. Department of Health and Human Services, February 1989). Weaker dilutions, e.g. 1:100 may be used, but may not be in accordance with the CDC recommendations.
- Rinse thoroughly and allow to dry before return to service.

## MACHINE LAUNDERING OR DRYING IS NOT RECOMMENDED

- The materials used to manufacture CPU150 and the performance of the fabric may be adversely affected by permitting a cleaning agent to remain in contact with the fabric for a prolonged duration or a failure to adequately rinse the agent from the fabric.

## Warranty

### Foam core:

Foam core is covered by a 2-year manufacture defect full replacement warranty. Compression up to 1" is not considered a warranty issue, but normal wear and tear.

### Cover:

Cover material is warranted for one (1) year against delamination. Zippers and handles are covered for one year.

